HoMEDICS  
THERA:P

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HoMedics Consumer Relations, Service Center Dept. 168  
3000 Pontiac Trail, Commerce Township, MI 48390

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HoMEDICS  
THERA:P

## Manual Inflate Blood Pressure Monitor

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Correo:  
HoMedics, Inc.  
Service Center  
Dept. 168  
3000 Pontiac Trail  
Commerce Township,  
MI 48390

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# TABLE OF CONTENTS

## INTRODUCTORY INFORMATION:

<b>Important Product Notices and Safety Instructions .....</b>	<b>3</b>
<b>About Blood Pressure .....</b>	<b>5</b>
<b>What is Blood Pressure?</b>	
<b>Why Measure Your Blood Pressure?</b>	
<b>Blood Pressure Standard.....</b>	<b>5</b>
<b>How This Blood Pressure Monitor Works .....</b>	<b>8</b>

## USAGE INFORMATION:

<b>Name/Function of Each Part .....</b>	<b>7</b>
<b>Measurement Procedure.....</b>	<b>11</b>
<b>Important Notes Regarding Your Blood Pressure Measurement.....</b>	<b>11</b>
<b>Care, Maintenance &amp; Cleaning .....</b>	<b>14</b>
<b>Specifications .....</b>	<b>14</b>
<b>2 Year Warranty .....</b>	<b>15</b>

## IMPORTANT PRODUCT NOTICES AND SAFETY INSTRUCTIONS

When using your blood pressure monitor, basic precautions should always be followed. Please read and follow all instructions and warnings before using this product. Save these instructions for future reference.

- Please note that this is a home healthcare product only and it is not intended to serve as a substitute for the advice of a physician or medical professional.
- Do not use this device for diagnosis or treatment of any health problem or disease. Measurement results are for reference only. Consult a healthcare professional for interpretation of pressure measurements. Contact your physician if you have or suspect any medical problem.  
Do not change your medications without the advice of your physician or healthcare professional.
- Proper cuff size is critical for accurate measurements. Follow the instructions in this manual and printed on the arm cuff to ensure the appropriate size of cuff is being used.
- Use blood pressure monitor only for its intended use.
- Do not wrap the cuff around body parts other than your arm.

- This product is intended for use in measuring the blood pressure of adults. Do not use this product on infants or those who cannot express their own consent. This product should not be operated by children.
- Do not overinflate cuff as this may cause injury and/or damage to the product.

Blood pressure measurements determined with this device are equivalent to those obtained by a trained observer using the cuff/stethoscope auscultatory method, within the limits prescribed by the American National Standard, Manual, electronic, or automated sphygmomanometers.

## ABOUT BLOOD PRESSURE

### What is Blood Pressure?

Blood pressure is the pressure exerted on the artery walls while blood flows through the arteries. The pressure measured when the heart contracts and sends blood out of the heart is systolic (highest) blood pressure. The pressure measured when the heart dilates with blood flowing back into the heart is called diastolic (lowest) blood pressure.

### Why Measure Your Blood Pressure?

Among today's various health problems, those associated with high blood pressure are very common. High blood pressure dangerously correlates with cardiovascular diseases. Therefore, blood pressure monitoring is important for identifying those at risk.

## BLOOD PRESSURE STANDARD

The **U.S. Department of Health and Human Services and the National Institutes of Health** have developed a blood pressure standard that identifies areas of low- and high-risk blood pressure. This standard, however, is a general guideline as an individual's blood pressure varies among different people, age groups, etc.

## Categories for Blood Pressure Levels in Adults

	Systolic (mmHg)	Diastolic (mmHg)
Stage 2 Hypertension	≥160	≥100
Stage 1 Hypertension	140~159	90~99
Prehypertension	120~139	80~89
Normal	<120	<80

For adults 18 and older who are not on medicine for high blood pressure, are not having a short-term serious illness, and do not have other conditions, such as diabetes and kidney disease.

To determine category of risk when systolic and diastolic readings fall into two areas, use the higher of the two numbers for classification.

There is an exception to the above definition of high blood pressure for people with diabetes and chronic kidney disease. A blood pressure of 130/80 mmHg or higher is considered high blood pressure for those individuals.

It is important that you consult with your physician regularly. Your physician will tell you your normal blood pressure range as well as the point at which you will be considered at risk.

For reliable monitoring and reference of blood pressure, keeping long-term records is recommended.

## NAME/FUNCTION OF EACH PART USING THE ARM CUFF



**Very Important: Proper cuff size is critical to accurate measurement.**

**This monitor is equipped with a Standard Size arm cuff that fits arms with circumferences of 8.25”–14.2” (21 cm – 36 cm). Measure the circumference of your upper arm to ensure that it falls within this range. If an alternative cuff size is needed, you may contact HoMedics Consumer Relations for more information.**

1. If the cuff is not assembled, pass the end of the cuff furthest away from the tubing through the metal D-ring in order to form a loop. The smooth side without the felt material should be on the inside of the cuff loop.


### ASSEMBLY INSTRUCTIONS

1. Insert tubes into stethoscope.
2. Insert tube into stethoscope microphone.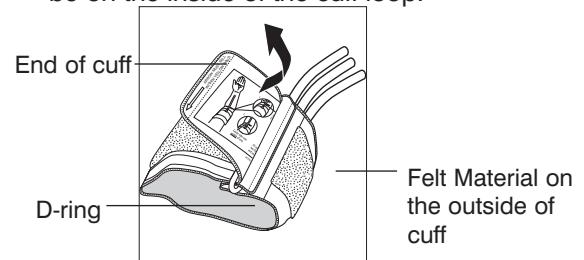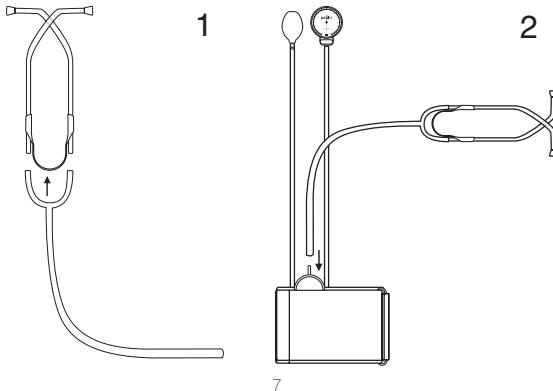


2. Open the arm cuff so that your arm may be placed through it.
3. Remove tight fitting clothing from around your upper left arm.


4. Position cuff edge 0.6-1.0 inches (1.5-2.5 cm) above elbow.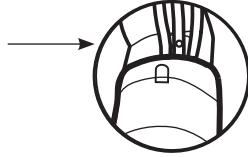

5. Pull the end of the cuff so that it tightens evenly around your arm. Press the hook and loop material together to secure. Allow room for 2 fingers to fit between the cuff and your arm.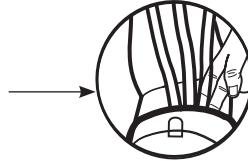

6. Lay your arm on a table (palm upward) so the cuff is at the same height as your heart. Make sure the tube is not kinked.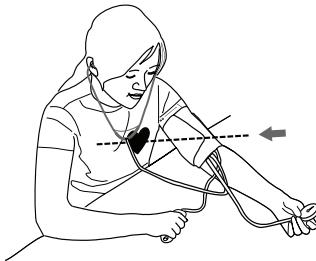

NOTE: If it is not possible to fit the cuff to your left arm, it can also be placed on your right arm. However, all measurements should be made using the same arm.

## MEASUREMENT PROCEDURE

### Important Notes:

- Blood pressure changes with every heartbeat and is in constant fluctuation throughout the day.
- Blood pressure measurement can be affected by the position of the user, his or her physiologic condition and other factors. For greatest accuracy, wait 1 hour after exercising, bathing, eating, drinking beverages with alcohol or caffeine, or smoking to measure blood pressure.
- Before measurement, it is suggested that you sit quietly for 15 minutes as measurements taken during a relaxed state will have greater accuracy. You should not be physically tired or exhausted while taking a measurement.
- During the measurement, do not talk or move your arm or hand muscles.

## IMPORTANT NOTES REGARDING YOUR BLOOD PRESSURE MEASUREMENT

- Take your reading in a comfortable environment as measurements can be affected by hot or cold temperatures. Take your blood pressure at normal body temperature.
- Do not move or talk during measurement as this can elevate readings.
- Do not move or cross legs during measurement. Keep feet flat on floor.
- Do not touch cuff or stethoscope microphone during measurement procedure.

- It is suggested that you take your measurements at the same time each day and use the same arm for consistency.
- Users should wait a minimum of 5 minutes before taking additional measurements. More time may be necessary depending upon your physiology.
- **The measurement results that users receive are for reference only. If users have any blood pressure concerns, please consult a physician.**

### 1. Position pressure cuff

- Whether sitting or lying, be sure that your arm rests at the same level as your heart.
- Turn the cuff so that the stethoscope microphone is on the inside of the arm just above the elbow. This locates it over the large artery near the hollow of the elbow. (This artery can be also found by feeling for the pulse of the artery.)
- Grasp the end of the cuff and pull it snug, then wrap it around and over your arm, pressing together with hook and loop tape to bind the cuff securely.


### 2. Inflate the cuff

- Feel the pulse of the artery with your fingertips to be sure the stethoscope microphone is directly over it.
- Insert the earpieces of the stethoscope (adjust by turning earpieces to fit comfortably in your ears).
- Hold the gauge in your left hand or clip onto the loop provided for reading.
- Close the air-flow valve on the bulb (turn clockwise). DO NOT OVERTIGHTEN.
- Inflate the cuff by repeatedly squeezing the bulb with your right hand.
- Listen to the pulse beat while inflating the cuff. Watch the gauge.
- When you can no longer hear the pulse beat, raise the pressure an additional 30mm.

### 3. Slowly deflate the cuff

- Slowly open the air-flow valve by turning counter clockwise so that the pressure drops 2-4mmHg with each beat of your heart. (This will usually mean a drop of one to two marks on the gauge every second.)
- The rate of deflation is important for accurate reading.

**CAUTION:** Do not leave cuff inflated for too long as the inflated cuff has shut off blood flow to the arm.

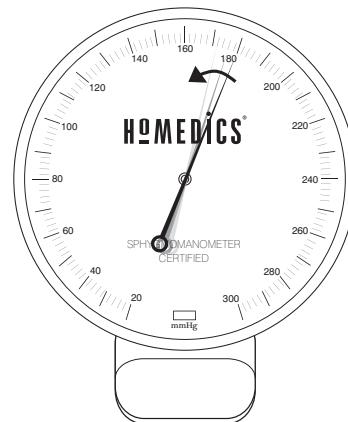#### 4. Determine Systolic pressure

- After opening the air-flow valve, listen carefully for a pulse beat. The moment you hear the faint rhythmic tapping or thumping sound of the pulse beat, note the reading on the gauge. This is your systolic (highest number) blood pressure.
- It may be different at first for you to detect the sounds of the pulse beat, listen carefully and acquaint yourself with these unfamiliar sounds. You should also notice the needle on the gauge “bounce” when your systolic pressure is reached.

#### 5. Determine Diastolic pressure

- Allow the pressure to continue dropping at the same rate as before 2-4mmHg per second. Listen carefully with the stethoscope. The sounds you hear will change phases.
- From the first sharp tagging or thud, they will soften to blowing or swishing sounds.
- Watch the falling needle. At the exact point when you can no longer hear the sounds, read the gauge. This is your diastolic (lower) blood pressure reading.

#### Care, Maintenance & Cleaning

1. Clean the blood pressure monitor body and cuff carefully with a slightly damp, soft cloth. Do not press. Do not wash cuff or use chemical cleaner on it. Never use thinner, alcohol or petrol (gasoline) as cleaner.
2. If the unit is stored near freezing, allow it to acclimate to room temperature before use.
3. This blood pressure monitor is not field service-able. You should not use any tool to open the device nor should you attempt to adjust anything inside the device. If you have any problems with this device, please contact HoMedics Consumer Relations (contact information can be found on the warranty page).
4. Do not immerse the unit in water as this will result in damage to the unit.

5. Do not subject the monitor and cuff to extreme temperatures, humidity, moisture, and direct sunlight. Protect from dust.
6. Do not fold the cuff and tubing tightly.
7. Do not disassemble the monitor or cuff. If in need of repair, refer to the warranty section of this manual.
8. Do not subject the monitor to extreme shocks (do not drop on floor).
9. Do not inflate the cuff unless wrapped around arm.
10. Do not wrap the cuff around body parts other than your arm.
11. Do not drop or insert any object into any opening or hose.
12. This monitor may not meet its performance specifications if stored or used outside of these temperature and humidity ranges:
  - Storage Environment
  - Operating Environment

#### SPECIFICATIONS

Operation Environment:	Temperature: 50°F~104°F (10°C ~40°C) Humidity: 40~85% RH max.
Storage Environment:	Temperature: 14°F~140°F (-10°C~60°C) Humidity: 10~90% RH max.
Arm Circumference:	Standard size adult arm cuff: 8.25"~14.2" (21 cm – 36 cm)

**Note:** These specifications are subject to change without notice.

**HoMEDICS**  
THERA:P

For service or repair, do not return this unit to the retailer. Contact HoMedics Consumer Relations at:

**Mail:**  
HoMedics, Inc.  
Service Center  
Dept. 168  
3000 Pontiac Trail  
Commerce Township, MI  
48390

**Email:**  
cservice@  
homedics.com

**Phone:**  
1-800-466-3342  
**Business Hours:**  
8:30am-5pm ET  
Monday-Friday

**Distributed by**  
**HoMEDICS**  
3000 Pontiac Trail  
Commerce Township, MI  
48390

## LIMITED TWO YEAR WARRANTY

HoMedics sells its products with the intent that they are free of defects in manufacture and workmanship for a period of five years from the date of original purchase, except as noted below. HoMedics warrants that its products will be free of defects in material and workmanship under normal use and service. This warranty extends only to consumers and does not extend to Retailers.

To obtain warranty service on your HoMedics product, mail the product and your dated sales receipt (as proof of purchase), postpaid, to the following address:

HoMedics Consumer Relations  
Service Center Dept. 168  
3000 Pontiac Trail  
Commerce Township, MI 48390

No COD's will be accepted.

HoMedics does not authorize anyone, including, but not limited to, Retailers, the subsequent consumer purchaser of the product from a Retailer or remote purchasers, to obligate HoMedics in any way beyond the terms set forth herein. This warranty does not cover damage caused by misuse or abuse; accident; the attachment of any unauthorized accessory; alteration to the product; improper installation; unauthorized repairs or modifications; improper use of electrical/power supply; loss of power; dropped product; malfunction or damage of an operating part from failure to provide manufacturer's recommended maintenance; transportation damage; theft; neglect; vandalism; or environmental conditions; loss of use during the period the product is at a repair facility or otherwise awaiting parts or repair; or any other conditions whatsoever that are beyond the control of HoMedics.

This warranty is effective only if the product is purchased and operated in the country in which the product is purchased. A product that requires modifications or adoption to enable it to operate in any other country than the country for which it was designed, manufactured, approved and/or authorized, or repair of products damaged by these modifications is not covered under this warranty.

**2** year  
limited warranty

**HoMEDICS**  
THERA:P

THE WARRANTY PROVIDED HEREIN SHALL BE THE SOLE AND EXCLUSIVE WARRANTY. THERE SHALL BE NO OTHER WARRANTIES EXPRESS OR IMPLIED INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS OR ANY OTHER OBLIGATION ON THE PART OF THE COMPANY WITH RESPECT TO PRODUCTS COVERED BY THIS WARRANTY. HoMEDICS SHALL HAVE NO LIABILITY FOR ANY INCIDENTAL, CONSEQUENTIAL OR SPECIAL DAMAGES. IN NO EVENT SHALL THIS WARRANTY REQUIRE MORE THAN THE REPAIR OR REPLACEMENT OF ANY PART OR PARTS WHICH ARE FOUND TO BE DEFECTIVE WITHIN THE EFFECTIVE PERIOD OF THE WARRANTY. NO REFUNDS WILL BE GIVEN. IF REPLACEMENT PARTS FOR DEFECTIVE MATERIALS ARE NOT AVAILABLE, HoMEDICS RESERVES THE RIGHT TO MAKE PRODUCT SUBSTITUTIONS IN LIEU OF REPAIR OR REPLACEMENT.

This warranty does not extend to the purchase of opened, used, repaired, repackaged and/or resealed products, including but not limited to sale of such products on Internet auction sites and/or sales of such products by surplus or bulk resellers. Any and all warranties or guarantees shall immediately cease and terminate as to any products or parts thereof which are repaired, replaced, altered, or modified, without the prior express and written consent of HoMedics.

This warranty provides you with specific legal rights. You may have additional rights which may vary from country to country. Because of individual country regulations, some of the above limitations and exclusions may not apply to you.

For more information regarding our product line in the USA, please visit: [www.homedics.com](http://www.homedics.com)

©2009 HoMedics, Inc. and its affiliated companies, all rights reserved. HoMedics is a registered trademark of HoMedics, Inc. and its affiliated companies.  
All rights reserved.  
IB-BPM010A

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©2009 HoMedics, Inc. and its affiliated companies, all rights reserved. HoMedics is a registered trademark of HoMedics, Inc. and its affiliated companies. All rights reserved. IB-BPM010A